



ACE螺絲/圖釘二合一組合・鈦骨板系列・再生膜、鈦網、骨粉系列、止血棉・再生膜專用器械・生物活化玻璃(合成人造骨)・ACE彎機/手動擴骨器・ACE劈、擴骨專用器械・Hu-Friedy牙周器械



固定鈦網與再生膜專用器械

ACE truFIX™

Complete Fixation System (Screws & Tacks)

螺絲/圖釘二合一組合





Titanium



Implant Specialty

植牙好幫手

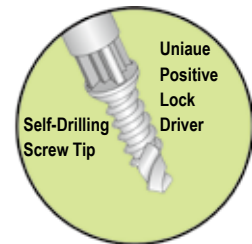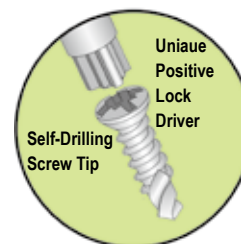

W-454-18015 truFIX Complete Fixation System (整組編號)

整組工具明細如下：

- W-454-19001 truFIX Driver Handle 握柄
- W-454-17001 truFIX Driver Tip 圖釘專用-手動接頭
- W-454-15001 truSCREW Driver Tip 螺絲專用-彎機接頭
- W-454-15201 truSCREW Driver Tip, Contra-Angle 螺絲專用-彎機接頭
- W-454-17901 truSCREW Driver Removal Sleeve
- W-50-27-00045 Pilot Bur, 0.45mm, 27mm Length, Contra-Angle
- W-454-40201 1.1mm Pilot Drill, Optional, CA Drill 圖釘專用鑽針
- W-454-281001 Step Drill(Lag Screw), 0.8, 1.6mm Steps, Contra-Angle
- W-454-50201 truSCREW Packaging Removal Tool
- W-20-1577-02 0.88mm External Hex Hand Driver
- W-454-18501 truFIX Tray 工具收納盒



W-454-19001



W-454-17001



W-454-15001



W-454-15201



W-50-27-00045



W-454-17901



W-454-40201



W-454-281001



W-454-50201




W-20-1577-02

Self-Drilling™ Bone Screw 自鑽式骨固定螺絲

(獨特設計，手術必備套件)

材質: Titanium alloy (Ti-6 Al-4VELI) (整組可高溫消毒)

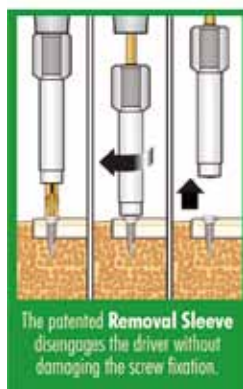

Titanium Mesh


鈦金屬網圓孔型、特殊型



W-003-9434



W-003-9429

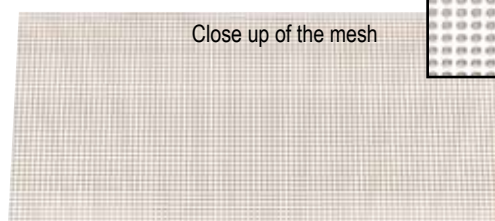

Close up of the mesh

W-003-9434 7 x 14mm, 0.2mm thick

W-003-9429 34 x 25mm, 0.1mm thick

W-003-9430 120 x 60mm, 0.1mm thick

W-003-9433 152 x 66mm, 0.2mm thick

1.2 TD truSCREW™

W-454-27202	truSCREW	3HD	1.2TD x 4.5TL	5/pkg
W-454-27203	truSCREW	3HD	1.2TD x 6.0TL	5/pkg
W-454-27204	truSCREW	3HD	1.2TD x 7.5TL	5/pkg
W-454-27205	truSCREW	3HD	1.2TD x 9.0TL	5/pkg
W-454-27206	truSCREW	3HD	1.2TD x 10.5TL	5/pkg

1.5 TD truSCREW™

W-454-27502	truSCREW	3HD	1.5TD x 6.0TL	5/pkg
W-454-27503	truSCREW	3HD	1.5TD x 7.5TL	5/pkg
W-454-27504	truSCREW	3HD	1.5TD x 9.0TL	5/pkg
W-454-27505	truSCREW	3HD	1.5TD x 10.5TL	5/pkg
W-454-27506	truSCREW	3HD	1.5TD x 12.0TL	5/pkg
W-454-27507	truSCREW	3HD	1.5TD x 13.5TL	5/pkg
W-454-27508	truSCREW	3HD	1.5TD x 15.0TL	5/pkg

2.5 TD truTACK™

W-96-003-13	truTACK	2.5HD	3mm TL	10/pkg
W-96-003-14	truTACK	2.5HD	5mm TL	10/pkg

truSCREW™



2.5 TD truTACK™



SCREW





獨特八角設計不易掉落，臨床使用更順手





ACE螺絲/圖釘二合一組合・鈦骨板系列・再生膜・鈦網、骨粉系列、止血棉・再生膜專用器械・生物活化玻璃(合成人造骨)・ACE彎機/手動擴骨器・ACE劈、擴骨專用器械・Hu-Friedy牙周器械


BONE SCREWS AND AUGMENTATION MESH

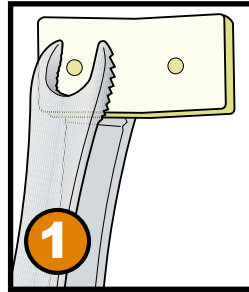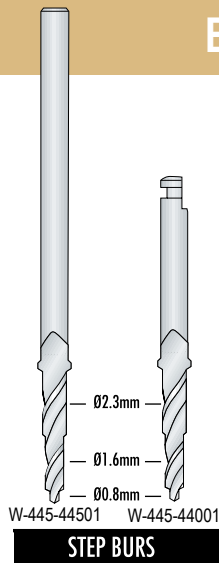


With the bone graft securely locked into position using the bone holding forceps, use the bone screw lag step drill (454-44001) to drill lag holes through the dimple markings on the graft.
夾骨鉗夾住骨塊鑽洞

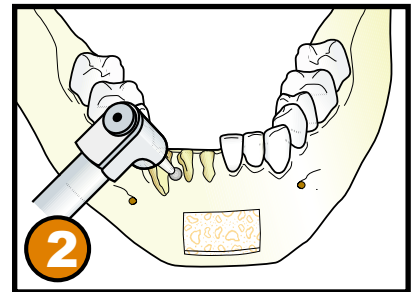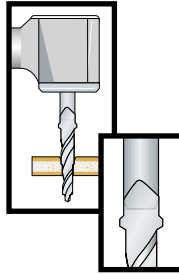



Using a round bur, decorticate the recipient site to initiate bleeding in the area.

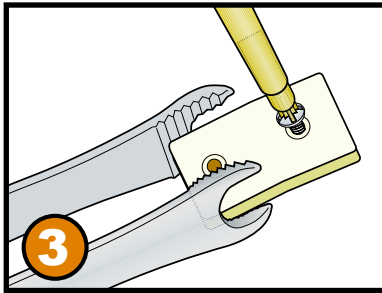

Insert the bone screw

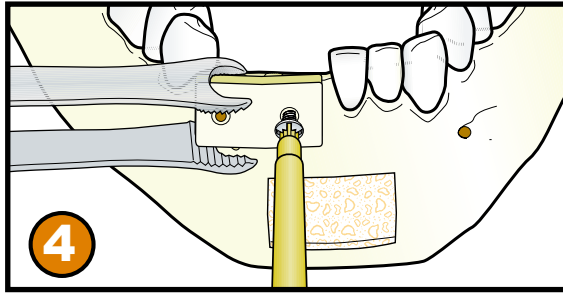

Screw the Self-Drilling Surgical Bone Screw into the recipient site.

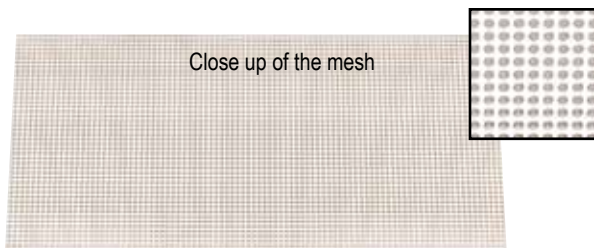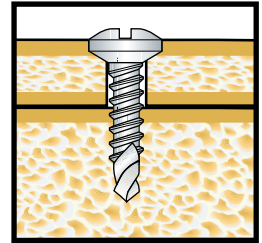

W-003-9430 120 x 60mm, 0.1mm thick
W-003-9433 152 x 66mm, 0.2mm thick


臨床操作圖




臨床操作圖




TUTORIAL TITANIUM MESH GRAFTING WITH ACE BONE SCREWS

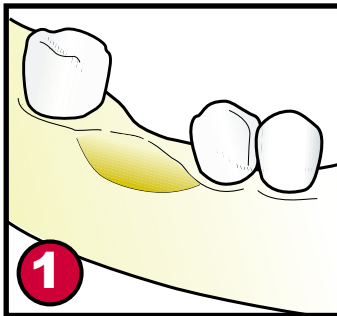


1
Periosteal flap elevated and defect exposed.

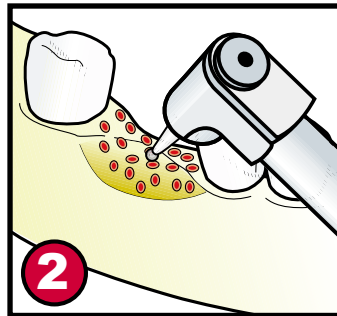


2
Examine the defect. Using a round bur, decorticate the recipient bone in the area and establish fresh bleeding.

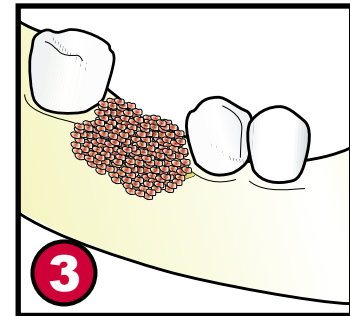


3
Fill the defect with the desired graft material and cover it with a titanium mesh and RCM⁶ Membrane.

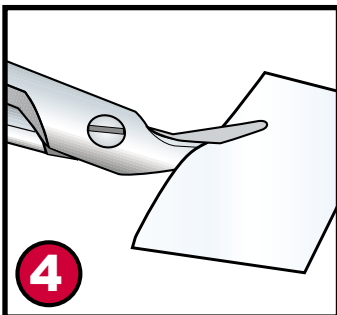


4
Cut the ACE RCM⁶ Resorbable Membrane to a defined geometry to conform to the defect area being restored.

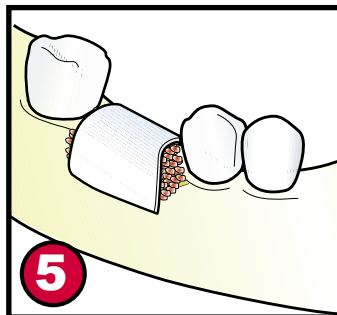


5
Drape the defect area with the ACE RCM⁶ Resorbable Membrane.

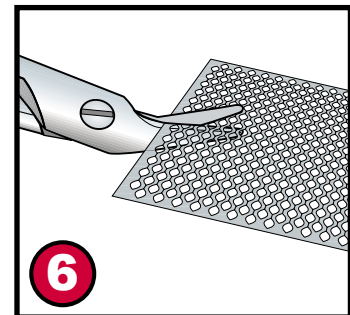

6
Cut the 0.1mm or 0.2mm thick titanium mesh to geometry slightly larger than the area of the RCM⁶ Resorbable Membrane.

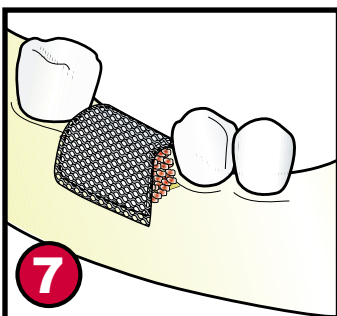

7
Drape the mesh over the defect area. Mark the mesh and host bone thru the mesh with an ACE step drill.

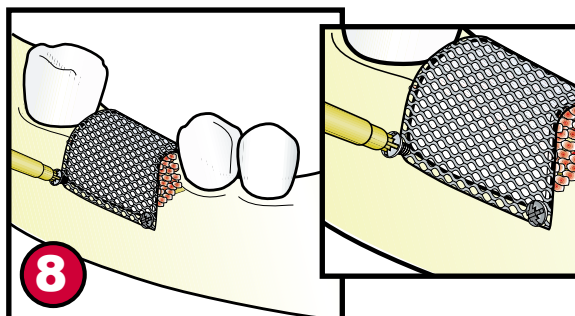


8
Secure the titanium mesh into the recipient bone using ACE Self-Drilling Bone Screws.

註：此病例流程以可吸收再生膜為例

臨床操作圖



使用ACE TruFix BoneScrew固定bone block

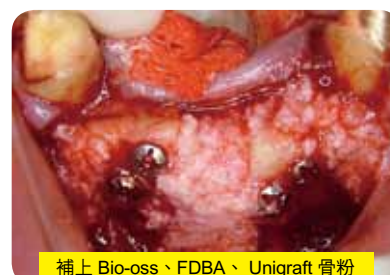

補上 Bio-oss、FDBA、Unigraft 骨粉



ACE螺絲/圖釘二合一組合・鈦骨板系列・再生膜、鈦網、骨粉系列、止血棉・再生膜專用器械・生物活化玻璃(合成人造骨)・ACE彎機/手動擴骨器・ACE劈、擴骨專用器械・Hu-Friedy牙周器械



Bone Screw Tacking Instruments

鈦合金圖釘組工具介紹(固定鈦網與再生膜專用)

材質：鈦合金

舊包裝

Titanium



Hexed for Placement and Removal

Threaded to Obtain Cortex Fixation Lock


獨特專利螺紋設計
讓鈦圖釘能更加與骨頭密合



鈦圖釘規格：

W-96-003-13 3mm Bone Screw Tacks, Titanium, 2.5mm Head Diam., 0.75 Shaft Diam. 10/Cassette

W-96-003-14 5mm Bone Screw Tacks, Titanium, 2.5mm Head Diam., 0.75mm Shaft Diam. 10/Cassette

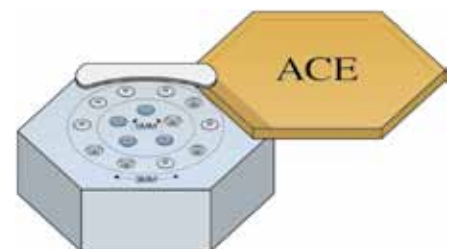

W-96-003-05 收納盒
可收納5mm圖釘 5支・3mm圖釘10支
Bone Screw Tack Cassette, Titanium,



W-96-003-15 Tack Seating Instrument, Titanium, Straight, Includes Cover 握柄(直)



W-96-003-16 Tack Seating Instrument, Titanium, Angled, Includes Cover 握柄(有角度)



W-64-003-12S Surgical Mallet, Stainless Steel 鎚子



W-20-1577-01 0.88mm External Hex Screwdriver, 73mm (Not Included In The Kit)
圖釘專用起子



W-50-44-00045 Twist Drill, 0.45mm diam., 4mm CL, HP, 44mm TL 鑽針(直機)




W-50-27-00045 Twist Drill, 0.45mm diam., 4mm CL, CA, 27mm TL 鑽針(彎機)

Bone Grafting Instruments

自鑽式骨固定螺絲組工具介紹



454-19501 Small Handle, Titanium, 3-1/2", 9cm 握柄-小

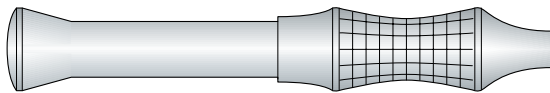

454-19001 Large Handle, Titanium, 4", 10cm 握柄-大

1.0mm Spline 1mm接頭適用於2.5mm HD(直徑)



454-15501 1mm Spline Driver Tip, 55mm 手動專用接頭-短



454-15502 1mm Spline Driver Tip, 85mm 手動專用接頭-長



454-15701 1mm Spline Driver Tip, CA, 19mm 接彎機手機專用接頭

1.25mm Spline 1.25mm接頭適用於3.0-4.0mm HD(直徑)



454-15001 1.25mm Spline Driver Tip, 55mm 手動專用接頭-短



454-15002 1.25mm Spline Driver Tip, 85mm 手動專用接頭-長



454-15201 1.25mm Spline Driver Tip, CA, 19mm 接彎機手機專用接頭


Bone Graft Holding Forceps 夾骨鑷



96-000-26 Bone Graft Holding Forceps, 8", 20cm

96-000-27 Bone Graft Holding Forceps, 6", 15cm

固定圖釘臨床操作圖

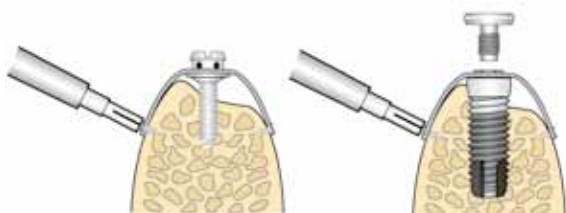

Available Screw Sizes

自鑽自攻式骨固定螺絲規格



Micro Screws - 1.2mm thread diameter - 包裝：5pc/set

2.5mm Head Diameter (Use the 1.0mm Spline Driver Tip)

454-21201 1.2mm thread diameter length 3.0mm

454-21203 1.2mm thread diameter length 6.0mm



Micro Screws - 1.5mm thread diameter - 包裝：5pc/set

2.5mm Head Diameter (Use the 1.0mm Spline Driver Tip)

454-21302 1.5mm thread diameter length 4.5mm

454-21303 1.5mm thread diameter length 6.0mm

454-21304 1.5mm thread diameter length 7.5mm

454-21305 1.5mm thread diameter length 9.0mm

454-21306 1.5mm thread diameter length 10.5mm



Mid Screws - 1.5mm thread diameter - 包裝：5pc/set

3.0mm Head Diameter (Use the 1.25mm Spline Driver Tip)

454-21501 1.5mm thread diameter length 4.5mm

454-21502 1.5mm thread diameter length 6.0mm

454-21506 1.5mm thread diameter length 12.0mm

454-21507 1.5mm thread diameter length 13.5mm

454-21508 1.5mm thread diameter length 15.0mm



Mini Screws - 2.0mm thread diameter - 包裝：5pc/set

3.0mm Head Diameter (Use the 1.25mm Spline Driver Tip)

454-22002 2.0mm thread diameter length 7.5mm

454-22003 2.0mm thread diameter length 9.0mm

454-22004 2.0mm thread diameter length 10.5mm

454-22005 2.0mm thread diameter length 12.0mm



Drill Point Tenting Screws

單支包裝

4.0mm Head Diameter (Use the 1.25mm Spline Driver Tip)

454-25501 1.5mm thread diameter length 7.0mm

454-25502 1.5mm thread diameter length 11.0mm

454-25503 1.5mm thread diameter length 15.0mm

454-26501 2.3mm thread diameter length 7.0mm

454-26502 2.3mm thread diameter length 11.0mm

Twist Drills 鑽針 -For 1.2/1.5mm Screws



50-25-0610 CA, 25mm, 0.7mm Head Diameter, Depth Stop 7mm

Step Pilot Drills 鑽針



454-44001 Step Pilot Drill, CA, SS, 30mm, (Step Diameter:0.8,1.6,2.1mm)



454-44501 Step Pilot Drill, HP, SS, 48mm, (Step Diameter:0.8,1.6,2.1mm)





ACE螺絲/圖釘二合一組合・鈦骨板系列・再生膜・鈦網・骨粉系列・止血棉・再生膜專用器械・生物活化玻璃(合成人造骨)・ACE彎機/手動擴骨器・ACE劈、擴骨專用器械・Hu-Friedy牙周器械



Micro Osseo-Plate System 鈦骨板系列

產品比例為1:1

Micro Osseo-Plates 使用1.2mm與1.5mm的Bone Screws

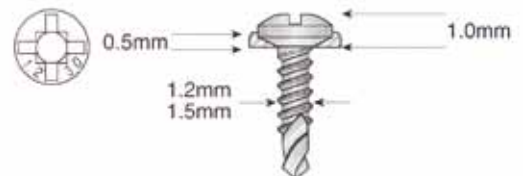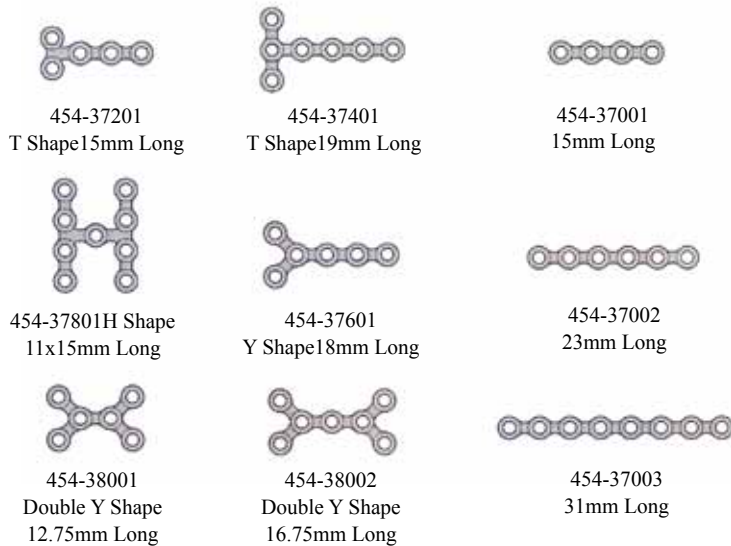

*ACE Titanium Self-Drilling Screws
for Micro Osseo-Plate System (0.5mm High)*



A27 頭顱模型



編號：5-U-3B-A27 十分解

編號：5-U-3B-A20 一般標準三分解

MESH PLATE



Titanium Mesh


鈦金屬網圓孔型、特殊型

12 point center hole locks on top of standard implant hexagon
12 point center hold locks to ACE Drill Point Tenting Screws
Can be secured with ACE Self-Drilling Bone Screws
Commercially pure titanium




A W-003-9434 7 x 14mm, 0.2mm thick

Close up of the mesh

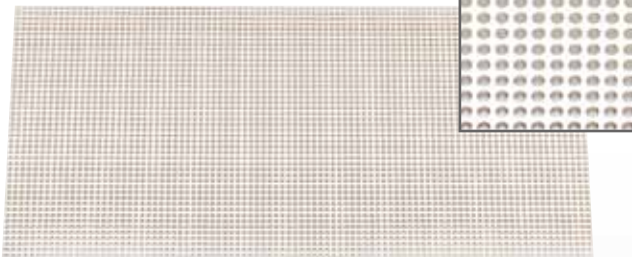

C W-003-9430 120 x 60mm, 0.1mm thick

D W-003-9433 152 x 66mm, 0.2mm thick

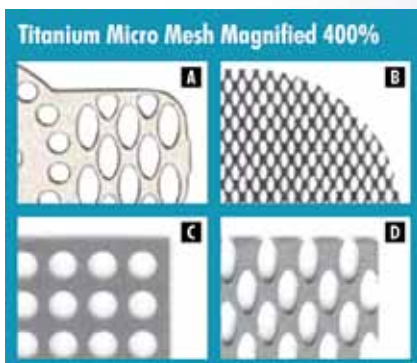

Fully resorbable
Easy to handle
Does not adhere to itself when hydrated
Zero memory
3-6 month predictable resorption






CYTOFLEX™

Precision Titanium Mesh

鈦網-小

B W-003-9429 34 x 25mm, 0.1mm thick (單片包裝)

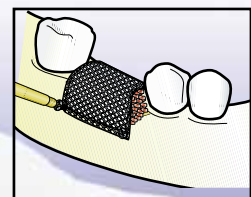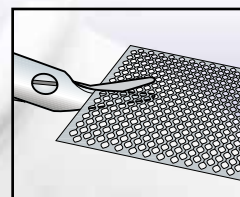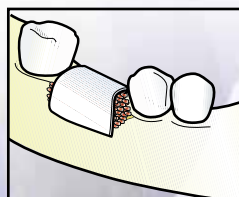

conFORM™
Resorbable Collagen MEMBRANE
ZERO MEMORY
Memorable Handling.




ACE螺絲/圖釘二合一組合・鈦骨板系列・再生膜、鈦網、骨粉系列、止血棉・再生膜專用器械・生物活化玻璃(合成人造骨)・ACE彎機/手動擴骨器・ACE劈、擴骨專用器械・Hu-Friedy牙周器械

TEF-GUARD™ Membrane 不可吸收再生膜

衛署醫器輸字第017451號

美國品牌・品質保證

TEF-GUARD™ Membrane

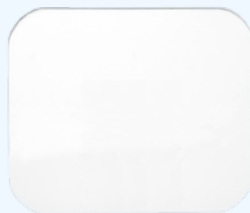

編號：TEF-002(平面)
規格：25 x 30mm (5片/盒)



編號：TEF-001T(凹面)
規格：12 x 24mm (5片/盒)



編號：TEF-002T(凹面)
規格：25 x 30mm (5片/盒)

臨床操作圖



術前PANO



植入IC5011STI Implant



放骨粉及不可吸收再生膜
(TEF-GUARD)



覆蓋再生膜



傷口無法完全縫合



一星期後拆線



手術後3個星期抽再生膜



再生膜特寫



再生膜抽出傷口特寫



抽出再生膜一星期後拆線



術後追蹤-傷口已癒合



術後PANO

META 拋棄式刮骨刀 義大利製



Cortical Bone Collector
拋棄式刮骨器(大型)

拋棄式刮骨器(小型)



ACE螺絲/圖釘二合一組合・鈦骨板系列・再生膜、鈦網、骨粉系列・止血棉・再生膜專用器械・生物活化玻璃(合成人造骨)・ACE彎機/手動擴骨器・ACE劈、擴骨專用器械・Hu-Friedy牙周器械

UNIGRAFT™ 生物活化玻璃(合成人造骨)

UNIGRAFT™ Bioactive glass Bone Graft Material 衛署醫器輸字第016661號



編號：P-E-U-2-6 (0.4g)
P-E-U-2-6 (1.0g)

規格：400 mg / 瓶 1000 mg / 瓶
Conveniently packaged in 0.4 gram vials,
5 per box, easy to use.

UNIGRAFT™ PUTTY

編號：CP-F-U-PUTTY0.5 (0.5cc)
CP-F-U-PUTTY1.0 (1.0cc)



Johnson & Johnson 快速止血紗布(可吸收性)

衛署醫器輸字第004219號

快速止血紗布可吸收性 (Surgicel)

編號：P-C-B1903GB (24片/盒)
P-C-B1903GB-1 (1片)



規格：5cm x 7cm

可溶性止血癒合紗布

衛署醫器製字第003479號

編號C-001：圓狀裝 0.5 x 0.7CM 60PCS/盒

編號C-002：塊狀裝 1.5 x 1.5CM 60PCS/盒



編號C-002



編號C-001



Bone Wax 骨臘

Made From Beeswax And Othe Natural Materials

衛署醫器輸字第006172號

2.5克 / 個

Small Membrane Punch 打洞專用再生膜鑷子

可打洞可定位



W-96-003-03 3", 7.5cm, Stainless Steel

Membrane Holding Forceps 夾取再生膜專用鑷子

Q-6862

W-96-003-22 Without Lock, Angled



W-96-003-24 With Lock

Membrane Placement Instrument



ACE Buser Type Elevator for Membrane Placement and Periosteal Elevation

W-03-001-39 7", 18cm



Q-3167 Raspatories, 24G, 17.5cm



Q-3171 BUSER, 17.5cm



ACE螺絲/圖釘二合一組合・鈦骨板系列・再生膜、鈦網、骨粉系列、止血棉・再生膜專用器械・生物活化玻璃(合成人造骨)・ACE彎機/手動擴骨器・ACE劈、擴骨專用器械・Hu-Friedy牙周器械

彎機/手動二合一擴骨器



ACE Ridge Expanding Kit

Implant Specialty

植牙好幫手



編號：W-0556-0000 (整組編號)



編號：LD





編號：V-186RF018-1




編號：U-S-SAW-DISC-231DC070
直徑：7 mm 長度：約22cm

整組工具明細：


- W-055-60201 2.5mm Bone Expander
- W-055-60301 3.1mm Bone Expander
- W-055-60401 3.6mm Bone Expander
- W-055-60501 4.3mm Bone Expander
- W-04-9734-01 Connection Driver Tool (Hex) for CA Handpiece
- W-96-002-28 Universal Ratchet
- W-20-1558-11 Driver Extender (Hex)
- W-96-004-49 Square Titanium Bowl
- W-20-1950-11 Autoclavable Tray 收納盒

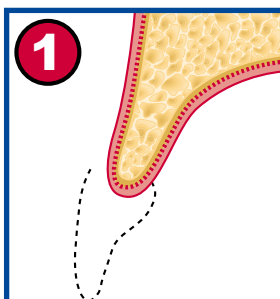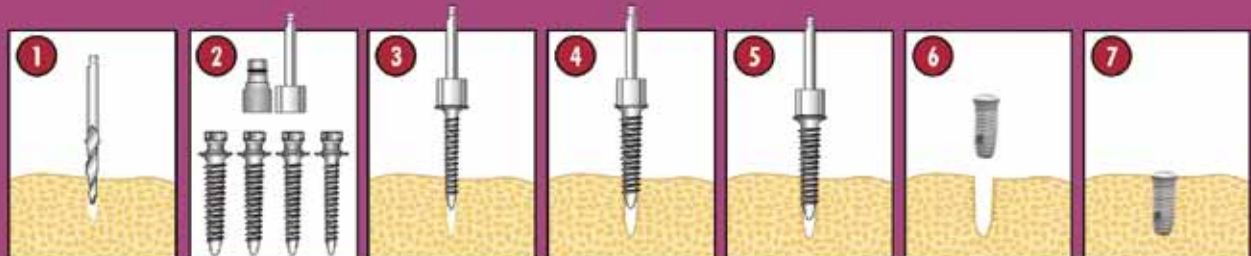

Thin anterior maxillary ridge requiring expansion.

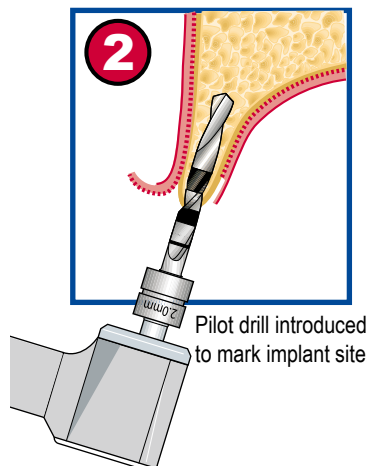

Pilot drill introduced to mark implant site.

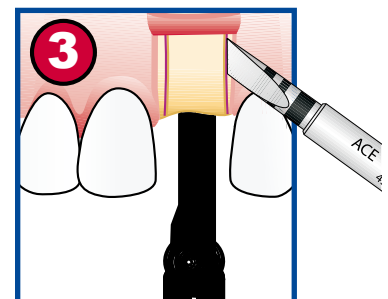

Vertical osteotomy cuts performed with ridge expanding chisel through the buccal cortex in an oblique fashion followed by horizontal osteotomy at ridge crest. Initial crest osteotomy can also be started with an ACE Mini Blade (786-6400).



劈骨專用器械組

ACE RIDGE EXPANDING CHISEL KIT

W-454-4700 Set of 4 Ridge Chisels 整組編號

整組工具明細：

W-454-4701 3.0mm Ridge Chisel

W-454-4801 4.0mm Ridge Chisel

W-454-4901 5.0mm Ridge Chisel

W-454-4501 9.5mm Ridge Chisel

W-765-1090 Autoclavable Tray 收納盒


W-454-1501 Chisel Handle (Uses 1.25mm Hex Driver) 蝴蝶扳手

All chisels and tray may be purchased separately.





Implant Specialty
植牙好幫手


Engage flate of osteotome with handle to allow widening and expanding bone split

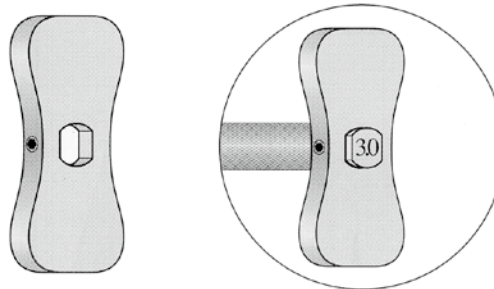


W-454-1501 蝴蝶扳手
Chisel Handle Optional, With
Attachment Screw
(Use 1.25mm Hex Driver)

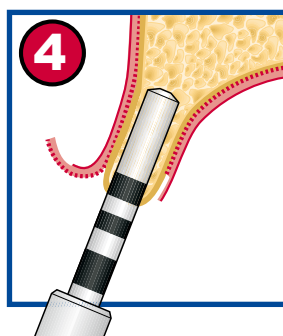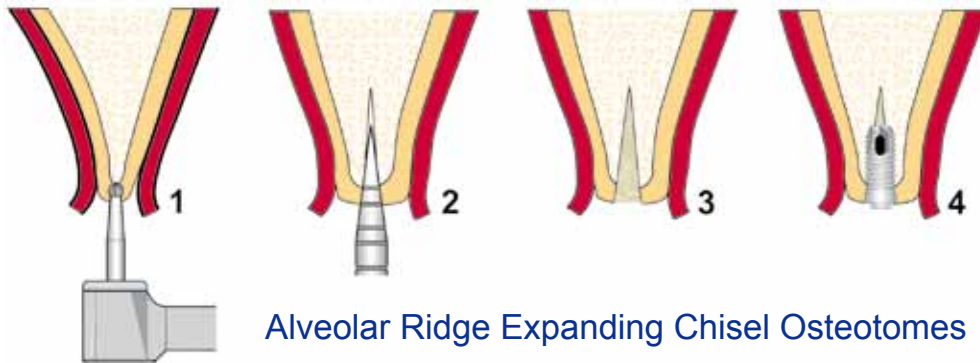

Ridge expansion
technique used to
complete implant
site.

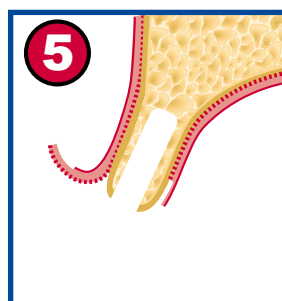

Complete implant osteotomy
site.

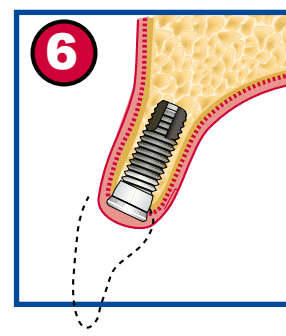

Implant in place; labial ridge
preserved.

EXPLORING NARROW RIDGES

彎機/手動 2用 擴骨組

MADE SIMPLE AND PRECISE

ACE




W-0556-0000 Kit Includes:

- | | | | |
|--------------|--|--------------|-----------------------|
| • 055-60201 | 2.5mm Bone Expander | • 96-002-28 | Universal Ratchet |
| • 055-60301 | 3.1mm Bone Expander | • 20-1558-11 | Driver Extender (Hex) |
| • 055-60401 | 3.6mm Bone Expander | • 96-004-49 | Square Titanium Bowl |
| • 055-60501 | 4.3mm Bone Expander | • 20-1950-11 | Autoclavable Tray |
| • 04-9734-01 | Connection Driver Tool (Hex)
for CA Handpiece | | |

Prepare the osteotomy with any number of surgical tools,
cutting to the depth of the desired implant.



劈/擴骨專用鋸片 (彎機手機專用)

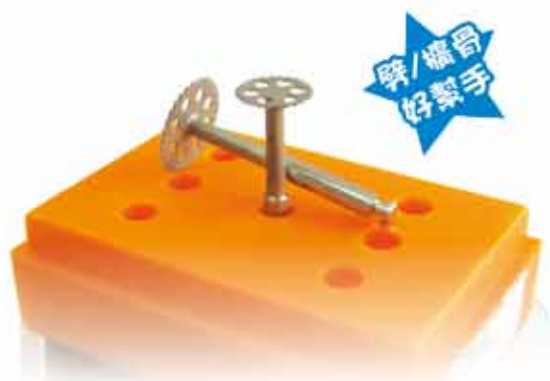


U-S-SAW-DISC-231DC070

直徑：7 mm
總長：約 22 mm




編號：LD

編號：V-186RF018-1

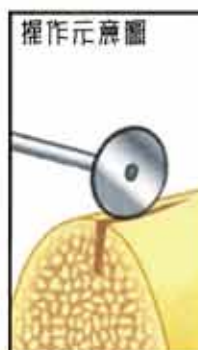



Bone Harvesting Instrument

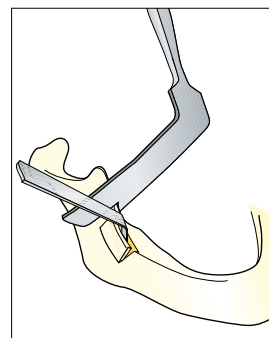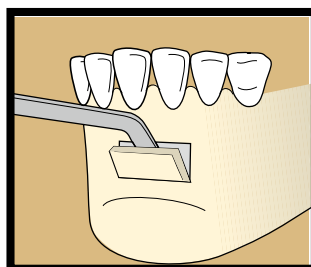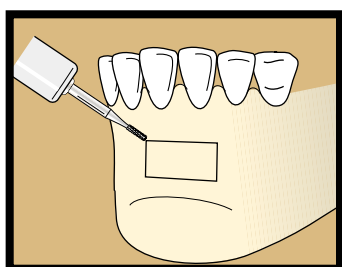

ACE Sagittal Split Osteotomy Chisels

取下顎骨專用側彎 Chisels

Useful in elevating the proximal segment

Implant Specialty

植牙好幫手



W-05-020-17 Cottle Nasal Chisel 6mm, 6-1/4", 16cm 微彎



側面微彎



W-05-010-18 8mm Chisel, Flat, 45° Angle, 6", 15cm

W-05-012-20 5mm Chisel, Flat, 45° Angle, 6", 15cm

Spatula Osteotomy Instruments



W-04-000-69 4mm x 1mm, Slight Curve, 8", 20cm



W-05-000-10 4mm x 30mm, 6", 長15cm 超薄 (厚度約0.45mm)




W-05-000-11 5mm x 35mm, 6", 長15cm 超薄 (厚度約0.75mm)



ACE螺絲/圖釘二合一組合・鈦骨板系列・再生膜、鈦網、骨粉系列、止血棉・再生膜專用器械・生物
活化玻璃(合成人造骨)・ACE彎機/手動擴骨器・ACE劈、擴骨專用器械・Hu-Friedy牙周器械

Periodontal Knives

牙周專用刮匙

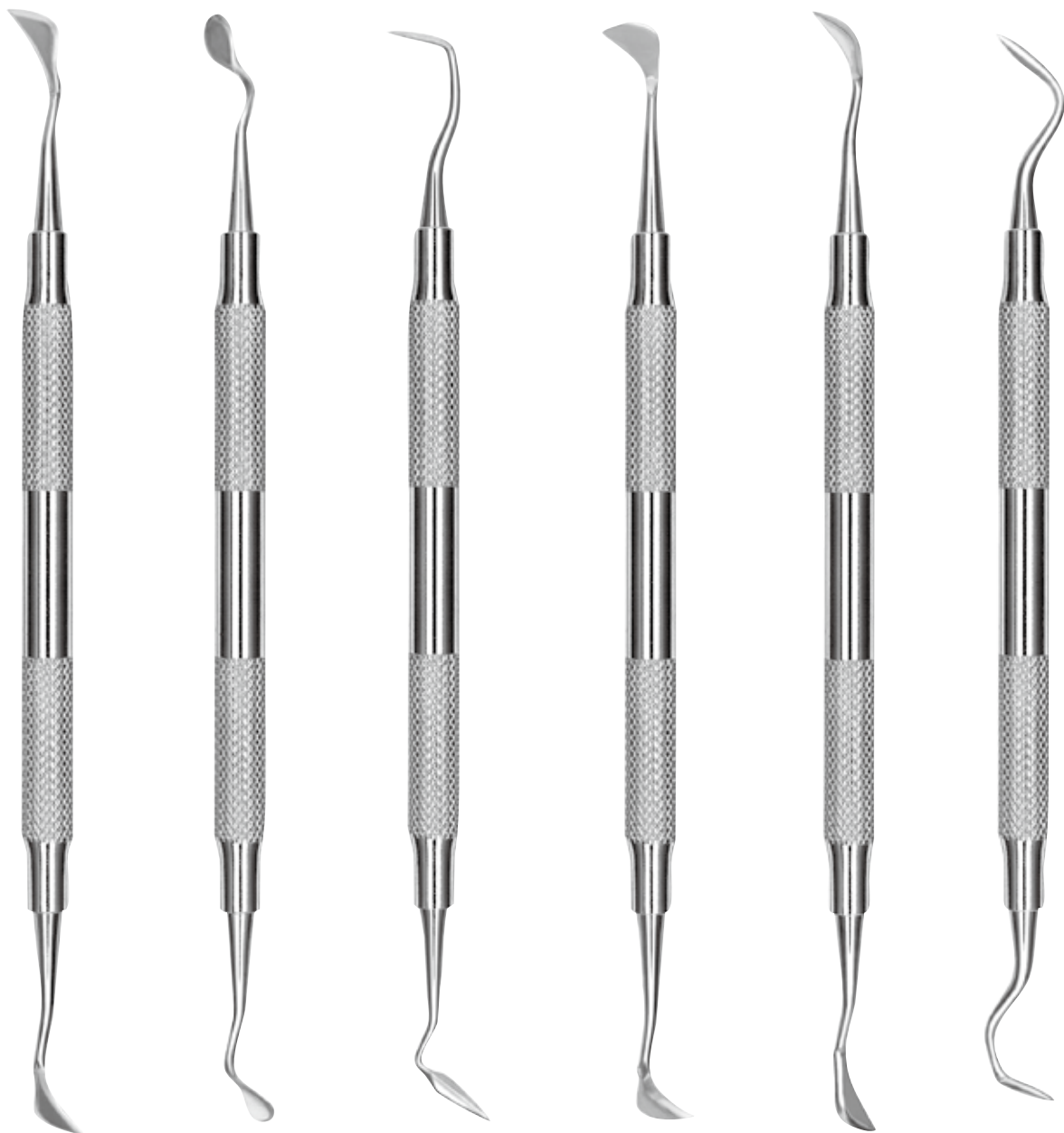

3-HU-KK15/16
W-417-9468
15K/16K
Kirkland

3-HU-KGF7
W-417-9469
7
Goldman-Fox

3-HU-KGF9
W-KGF9
9
Goldman-Fox

3-HU-K3/4
W-417-9479
3/4
Buck

3-HU-KSOLT1/2
W-KSOLT1/2
1/2
Solt


3-HU-KS01/2
W-417-9487
1/2
Orban

另備有Hu-Friedy相關器械目錄，歡迎來電洽詢

Periodontal Knives

牙周專用刮匙

Hu-Friedy®

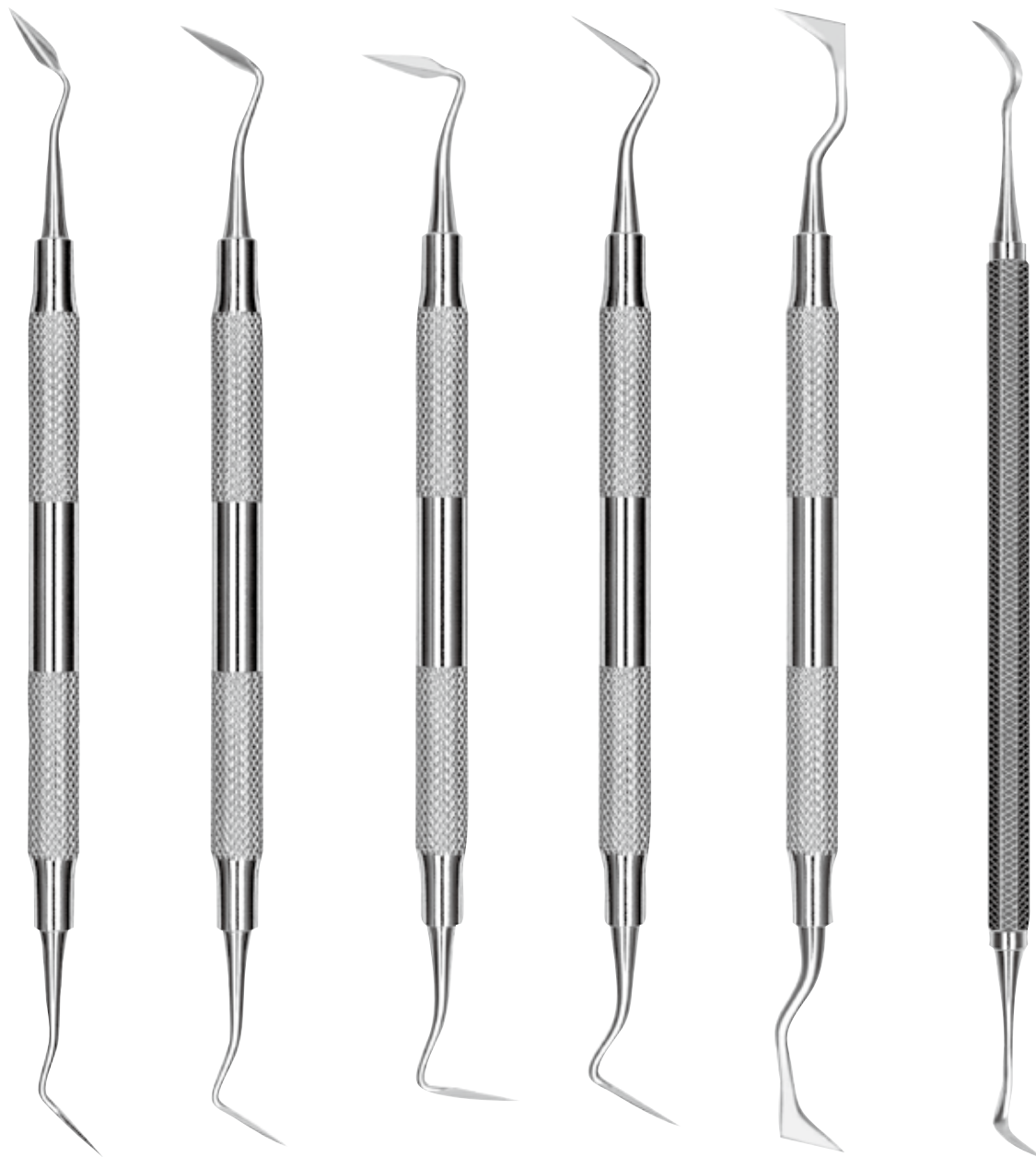

3-HU-KGF8
W-417-9470
8
Goldman-Fox

3-HU-KGF11
W-417-94729
11
Goldman-Fox

3-HU-KB5/6
W-417-9480
5/6
Buck

3-HU-KS1/2
W-417KS12
1/2
Sanders

3-HU-KUSC
W-417KUSC
USC
Towner 19/20

3-HU-KCK3
W-417-9481
CK-3
Ctane Kaplan


另備有Hu-Friedy相關器械目錄，歡迎來電洽詢



ACE螺絲/圖釘二合一組合・鈦骨板系列・再生膜、鈦網、骨粉系列、止血棉・再生膜專用器械・生物
活化玻璃(合成人造骨)・ACE彎機/手動擴骨器・ACE劈、擴骨專用器械・Hu-Friedy牙周器械

Periodontal Files

牙周專用刮匙

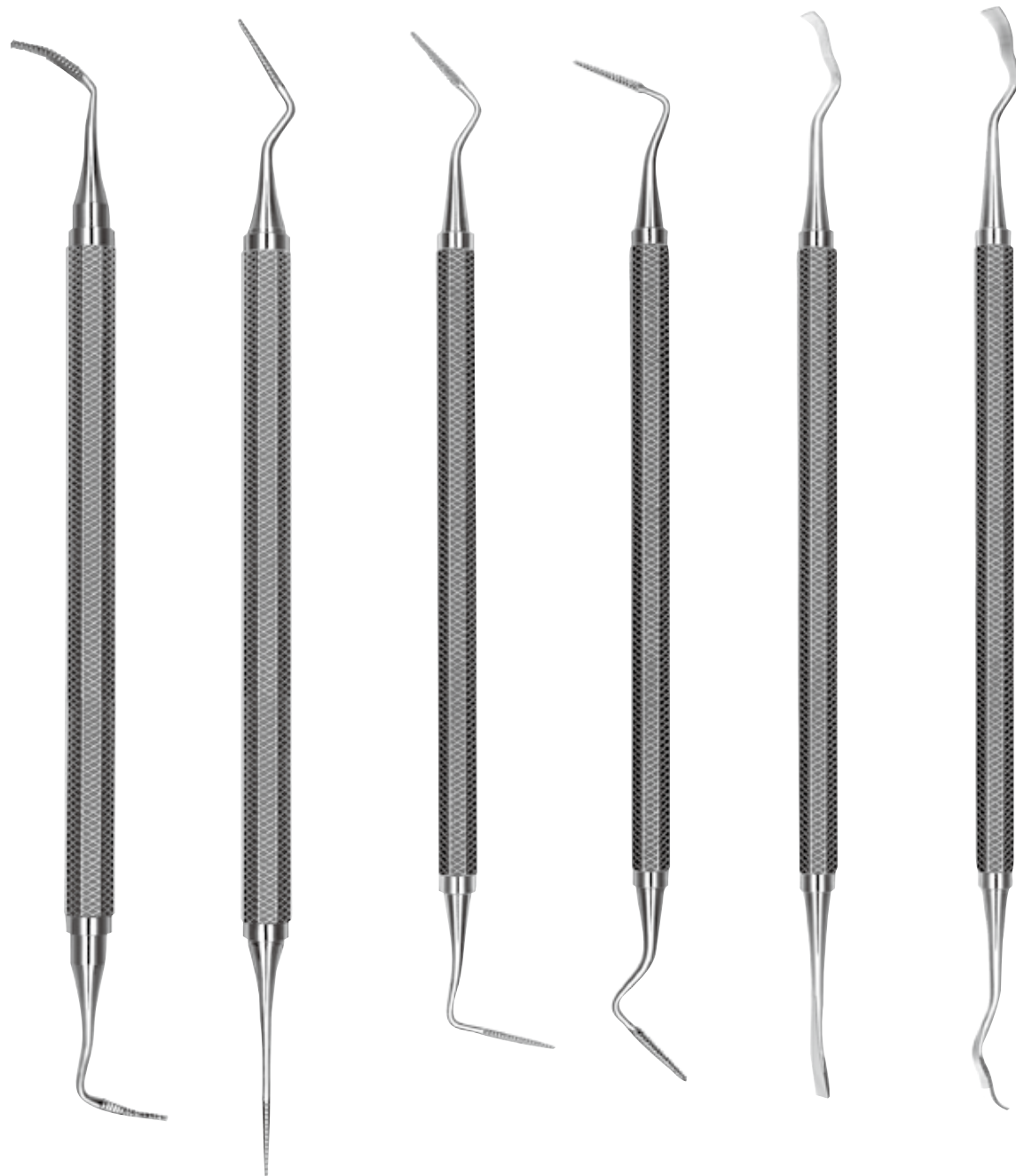

3-HU-FS9/10S

W-417-9432

9/10

Schluger Curved File

3-HU-FB11/12

W-417-9435

11/12

Buck Straight File

3-HU-FS1/2S

W-417-9438

1S/2S

Sugarman File M/D

3-HU-FS3/4S

W-417-9439

3S/4S

Sugarman File B/L

3-HU-S13K/TG

W-417-9463

13K/TG

3-HU-S13K/13KL

W-417-9462


13K/13KL

另備有Hu-Friedy相關器械目錄，歡迎來電洽詢

Periodontal Chisels

牙周專用刮匙

Hu-Friedy®

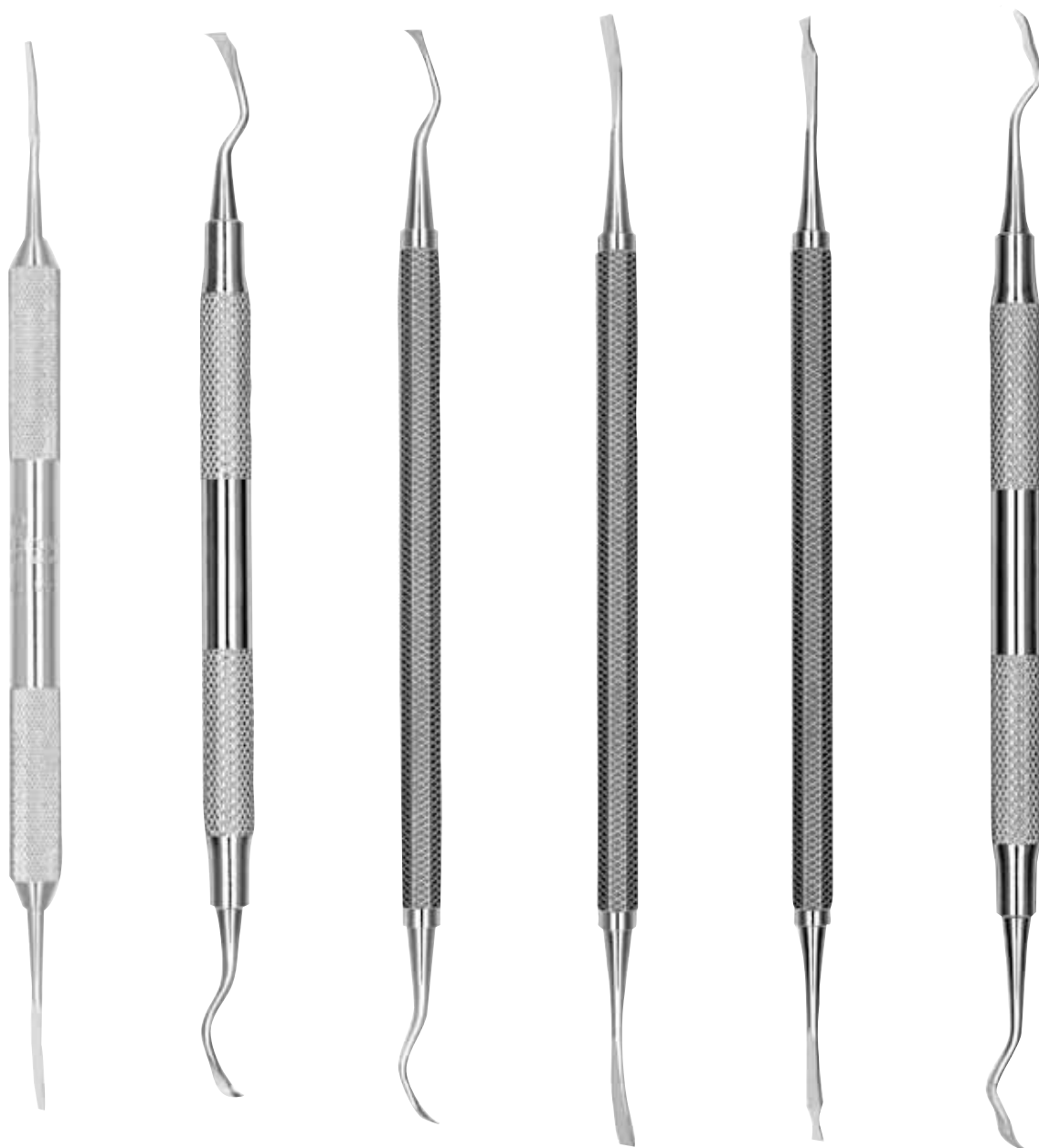

3-HU-CKN1/2
W-CKN1/26
1/2
Kramer-Nevins

3-HU-C36/37
W-417-9601
36/37
Rhodes Back Action

3-HU-S44/45
W-S44/45
44/45
Remington Hos

3-HU-CTG
W-417-9602
TG

3-HU-CTG
W-417-9603
TG-0
Modified w/Cutouts

3-HU-CSOLT3/4
W-CSOLT3/4
3/4 Solt

另備有Hu-Friedy相關器械目錄，歡迎來電洽詢



ACE螺絲/圖釘二合一組合・鈦骨板系列・再生膜、鈦網、骨粉系列、止血棉・再生膜專用器械・生物
活化玻璃(合成人造骨)・ACE彎機/手動擴骨器・ACE劈、擴骨專用器械・Hu-Friedy牙周器械

Periotome Blades

牙周病專用剝離器

立即拔牙植牙手術專用器械好幫手！



Periodontal Instruments

植牙周圍炎專用刮匙 (可替換)



3-HU-CIGFT4
#4
Extra-Flex

3-HU-CIGFT3
#3
Extra-Flex



W-IMPLACAR16
4R/4L
Columbia

W-417-4411
H6/H7
Plasteel Tip/Handle
(一組有配10對/組)



3-HU-IC4R/4L
W-417-4403
4R/4L(10對/組)
Plasteel Tips 塑膠材質



3-HU-I204S
W-I204S
I204S(10對/組)
Plasteel Tips 塑膠材質



3-HU-IH6/7
W-IH6/7
H6/H7(10對/組)
Plasteel Tips 塑膠材質



3-HU-IMPHDL6
W-417-4406
double-end
#6
Satin Steel

另備有Hu-Friedy相關器械目錄，歡迎來電洽詢



Probe

3-6-9-12



12-9-6-3

3-HU-PQ2N
Q-2N
Nabers
Color coded

Gracey Curette



3-HU-SG13/14
13/14



3-HU-SG15/16
15/16



3-HU-SG3/4
3/4



3-HU-SG7/8
7/8



3-HU-SYG7/8
YG7/8
Younger Good

磨刀石



3-HU-SS3C
No.3
Ceramic
Medium grit^{w/d}
7.6 x 3.0 x 0.6mm

另備有Hu-Friedy相關器械目錄，歡迎來電洽詢